



USER GUIDE

Using the TrioClearTM app

START YOUR TRIOCLEAR™ JOURNEY

Let's begin your smile makeover journey. Now you can download the TrioClear™ Aligner mobile app to discover key features such as:


1. Record your aligner wear time daily.
2. Capture your oral photos for progress updates to your doctor.
3. Discover more transformed smiles within our TrioClear™ community.
4. Receive reminders and notifications.

TRIOCLEAR™ ALIGNER
MOBILE APP IS NOW
AVAILABLE ON:




TABLE OF CONTENTS



OPEN APP AND LOGIN.....	04
HOMEPAGE AND CASE MANAGEMENT.....	05
PATIENT MANAGEMENT.....	07
PATIENT INFO. AND CASE LOG.....	07
TREATMENT PLAN.....	08
PHOTOS RECORDED BY CLINIC.....	10
PHOTOS RECORDED BY PATIENT / TRACKING DATA BY PATIENT.....	12
NEW CASE/ REFINEMENT.....	13
STANDARD PHOTOS TAKING FLOW IN NEW CASE REFINEMENT.....	14
NEW CASE REFINEMENT.....	16
GALLERY.....	18
SETTINGS - MY APP ACCOUNT.....	19

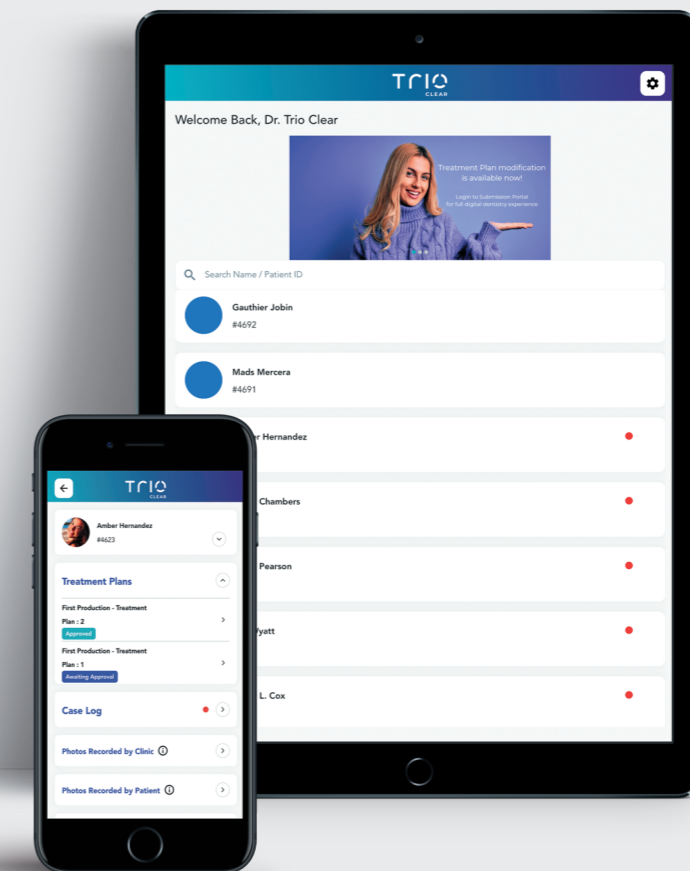

OPEN APP AND LOGIN

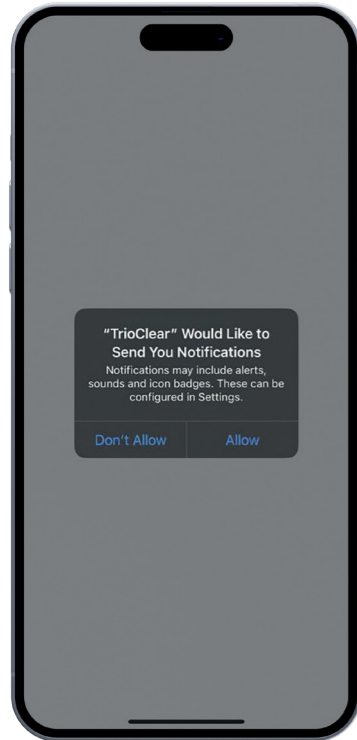

Allow Notifications to receive updates from case log

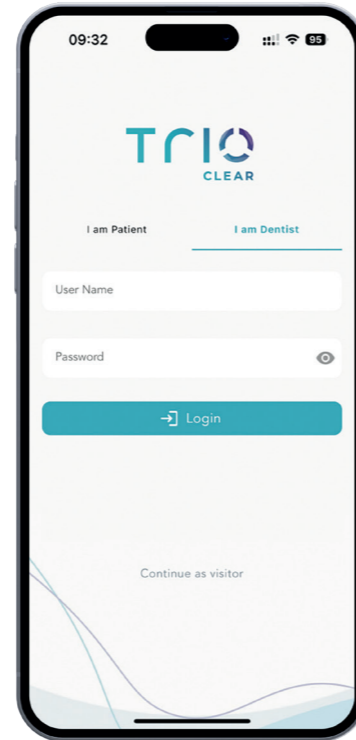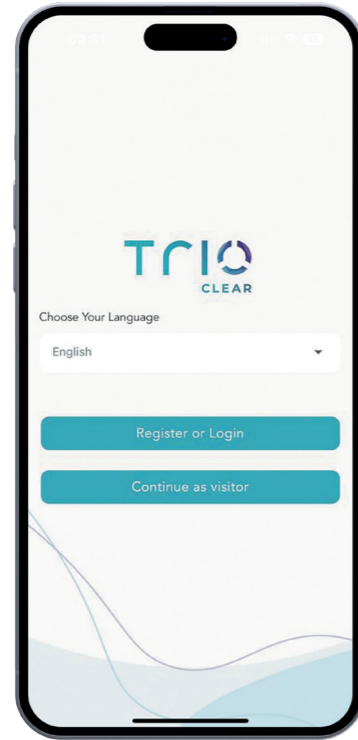


Choose "I am Dentist" and login

Login & Password = submission portal login set in real cases

HOMEPAGE AND CASE MANAGEMENT

HOME

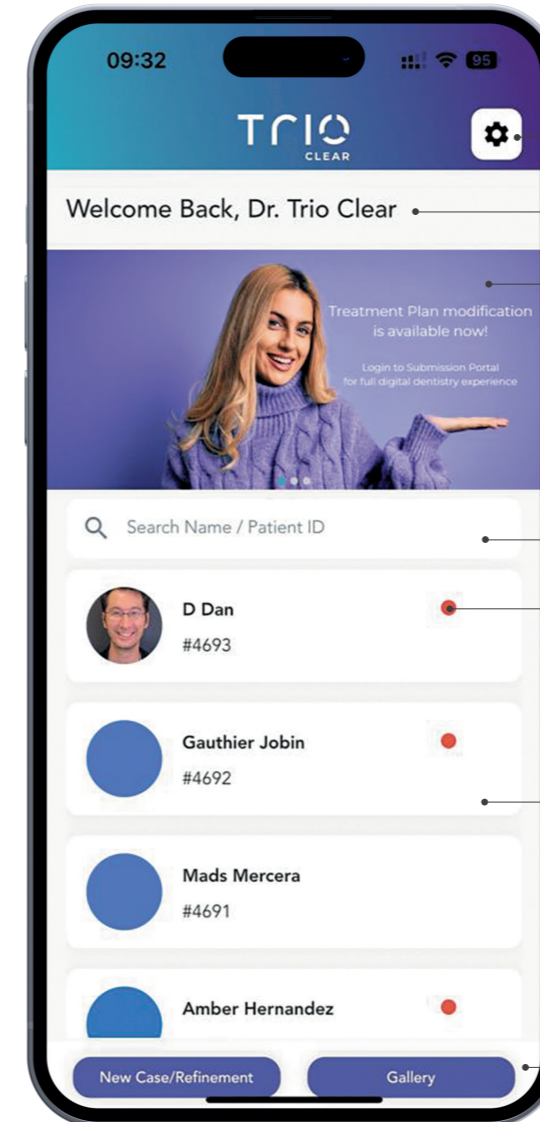

Settings button

Same doctor's name registered on submission portal

Top Banner:
 · Can be localized
 · Can land to specific webpage

Search bar

Notification dot

You can see all cases on submission portal

Buttons of:
 · New Case / Refinement
 · Gallery


TRIOCLEAR™ MOBILE APP CAN DO:

- Top Banner: Better option for marketing purpose
- View case info. / patient info.
- View treatment plan / case log
- Approve treatment plan
(Cannot modify treatment plan at mobile interface)
- Snap & upload patient's photos to submission portal
- Receive and track patient's voluntary photo uploads during the treatment
- Receive and track patient's voluntary uploads about their aligner wear time


PATIENT MANAGEMENT

Patient information and case log

PATIENT INFORMATION

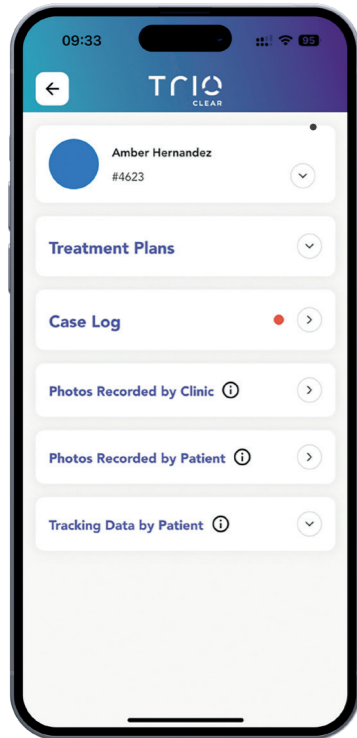

PATIENT INFORMATION

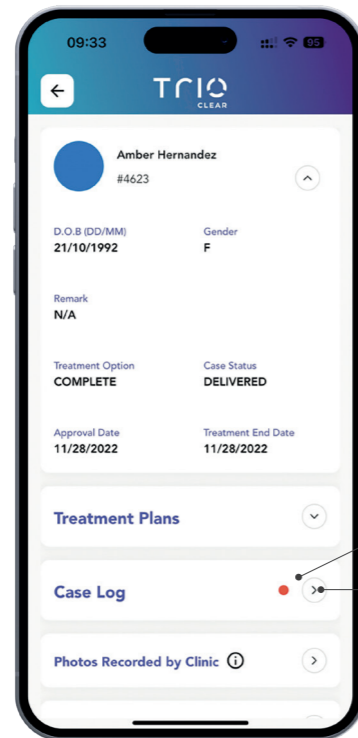


Notification dot
Sync to submission portal

Open case log


PATIENT MANAGEMENT

Treatment plan


TREATMENT PLAN DETAIL

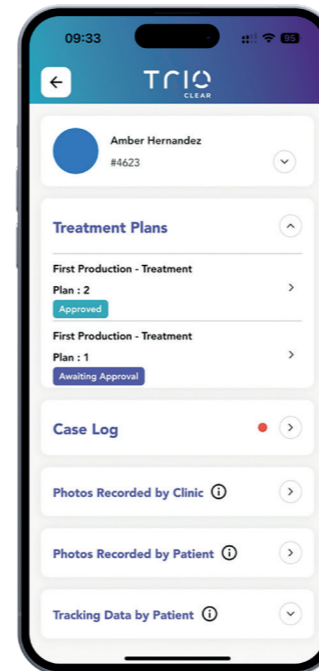

Show the **treatment plan** record & status

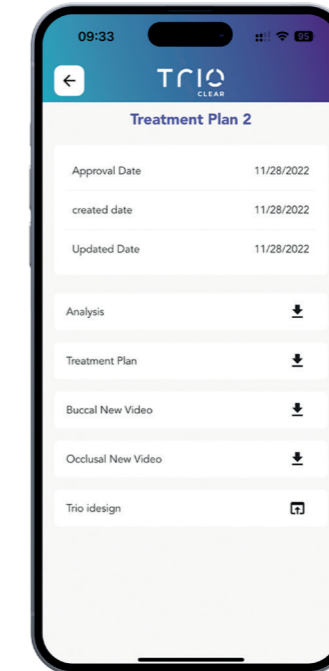


Show same set of **documents** in **treatment plan**

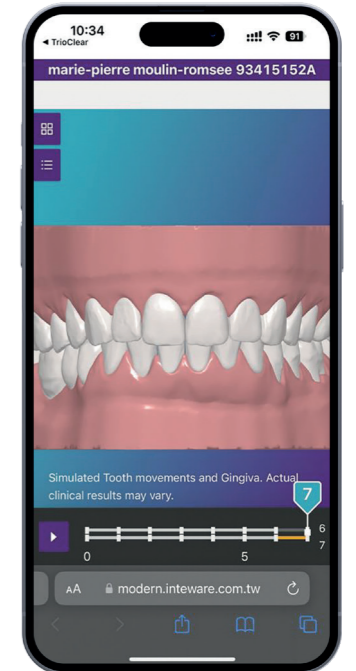

Can link to **TrioClear iDesign** Mobile version (Cannot modify plan)


CASE LOG

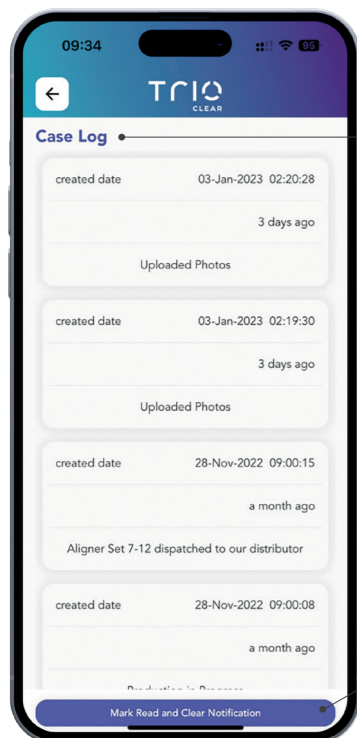


Show full case log
Same as submission portal

Mark Read and Clear Notification
Sync to submission portal

APPROVE TREATMENT PLAN

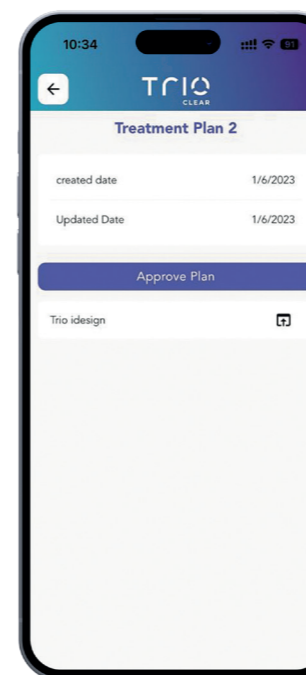

Approve treatment plan is available

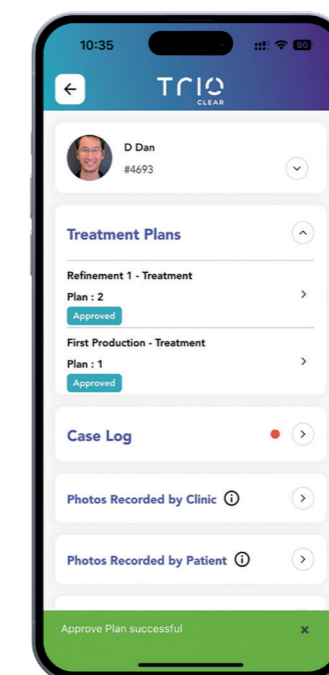

PATIENT MANAGEMENT

Clinical photo uploader feature

PRE-TREATMENT PHOTO'S



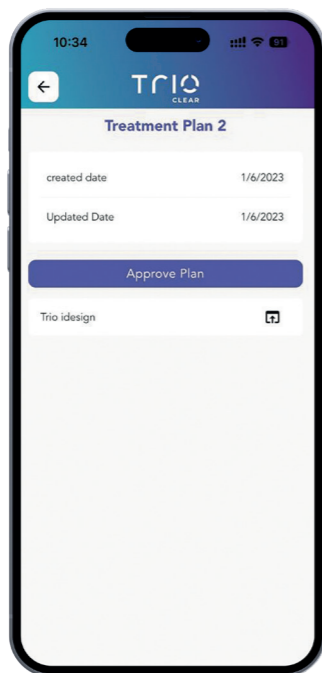


Photo sets taken by clinic

PRE-TREATMENT PHOTO'S

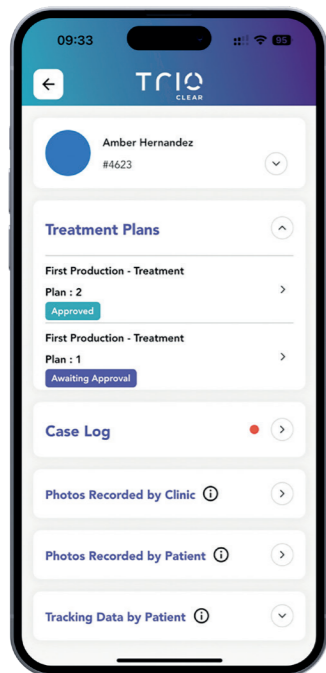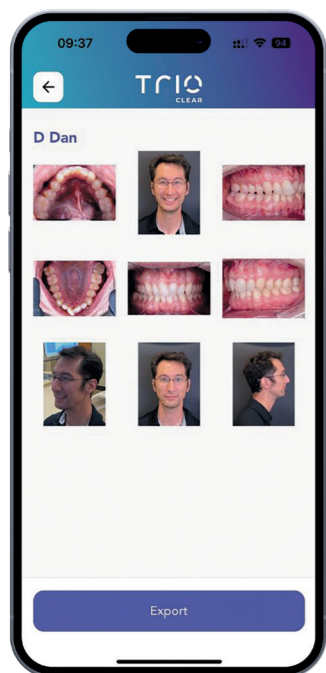

ADD NEW PHOTOS


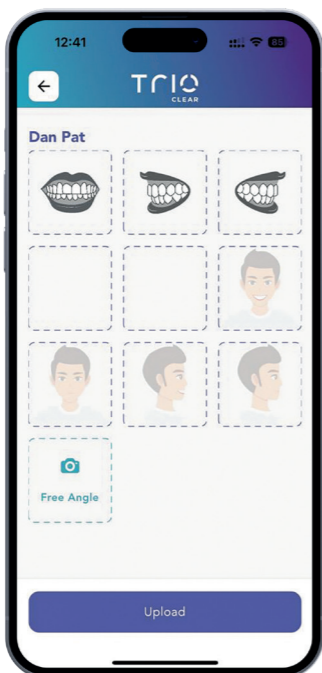


Photo taking process

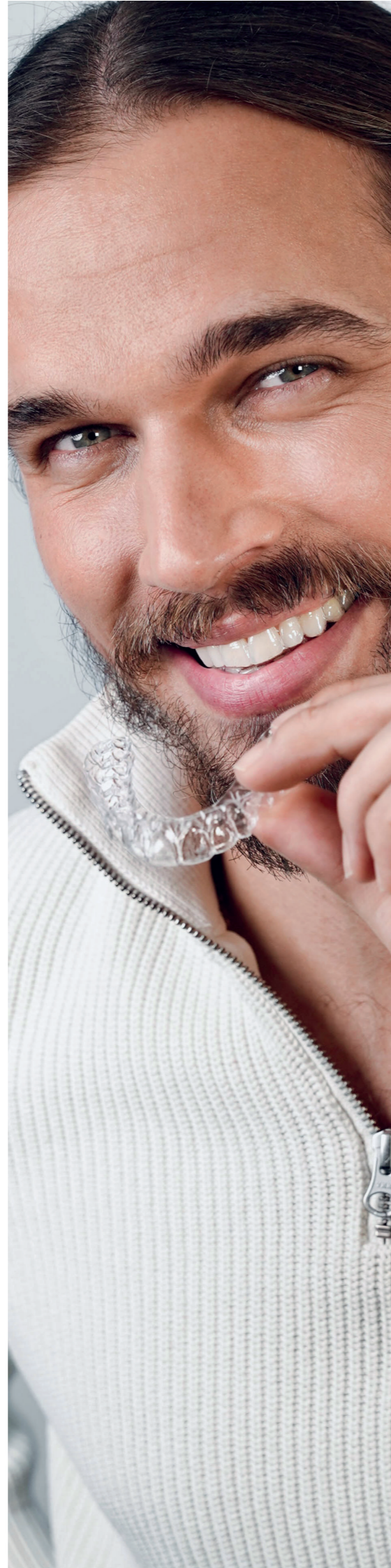

PATIENT MANAGEMENT

Photos recorded by clinic

Submission Portal -> Patient -> View photos



Photos can be found in "View Photos" on submission portal For internal reference only.



PATIENT MANAGEMENT


Photos recorded by patient / Tracking Data by Patient

PHOTOS RECORDED BY PATIENT

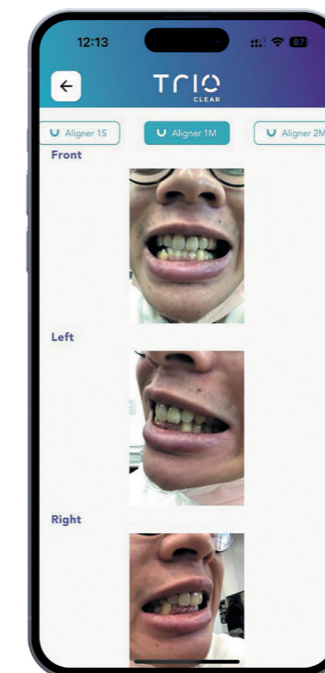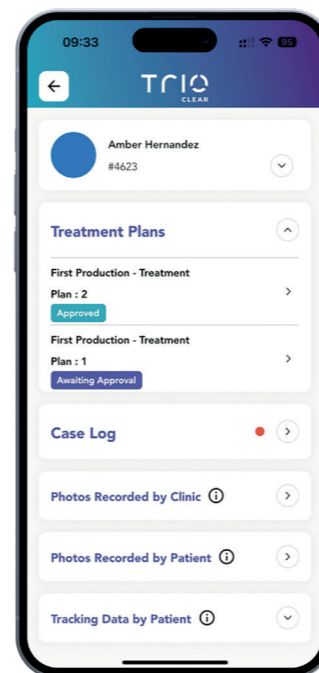


Show Patient's **uploads** for front, left & right view at different **aligner steps**

TRACKING DATA BY PATIENT

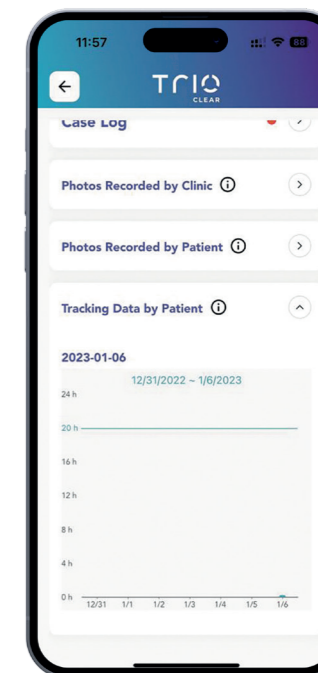


Show Patient's **uploads** of their aligner **wear time**


NEW CASE / REFINEMENT

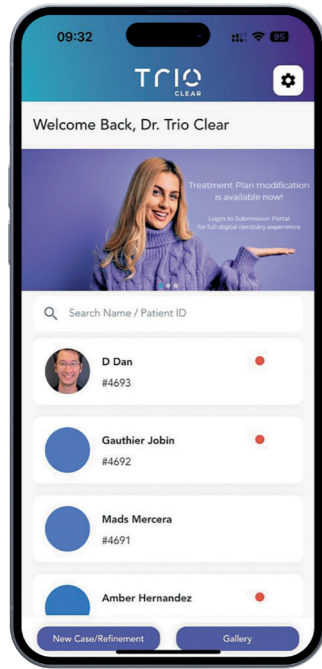

Choose "New case / Refinement"

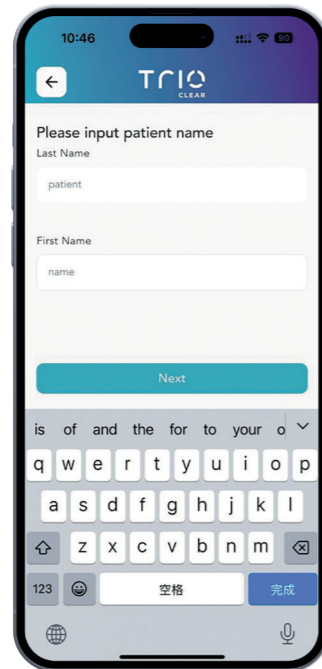

Input patient's name to create photo folder

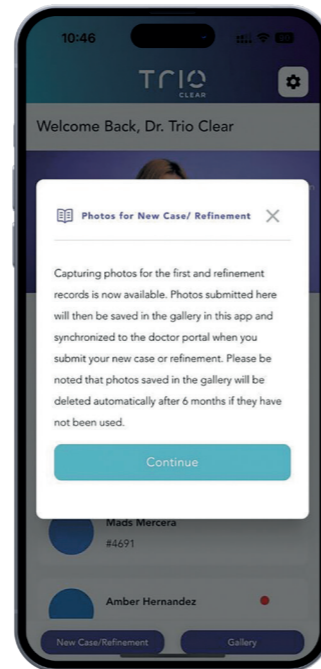



Reminder

- By using TrioClear App, you can simply take photos for new case & refinement and upload them to Doctor's Submission Portal directly
- Create case, choose the photo folder, submit the case on Doctor's Submission Portal

STANDARD PHOTOS

taking flow in new case refinement

MANDATORY PHOTOS

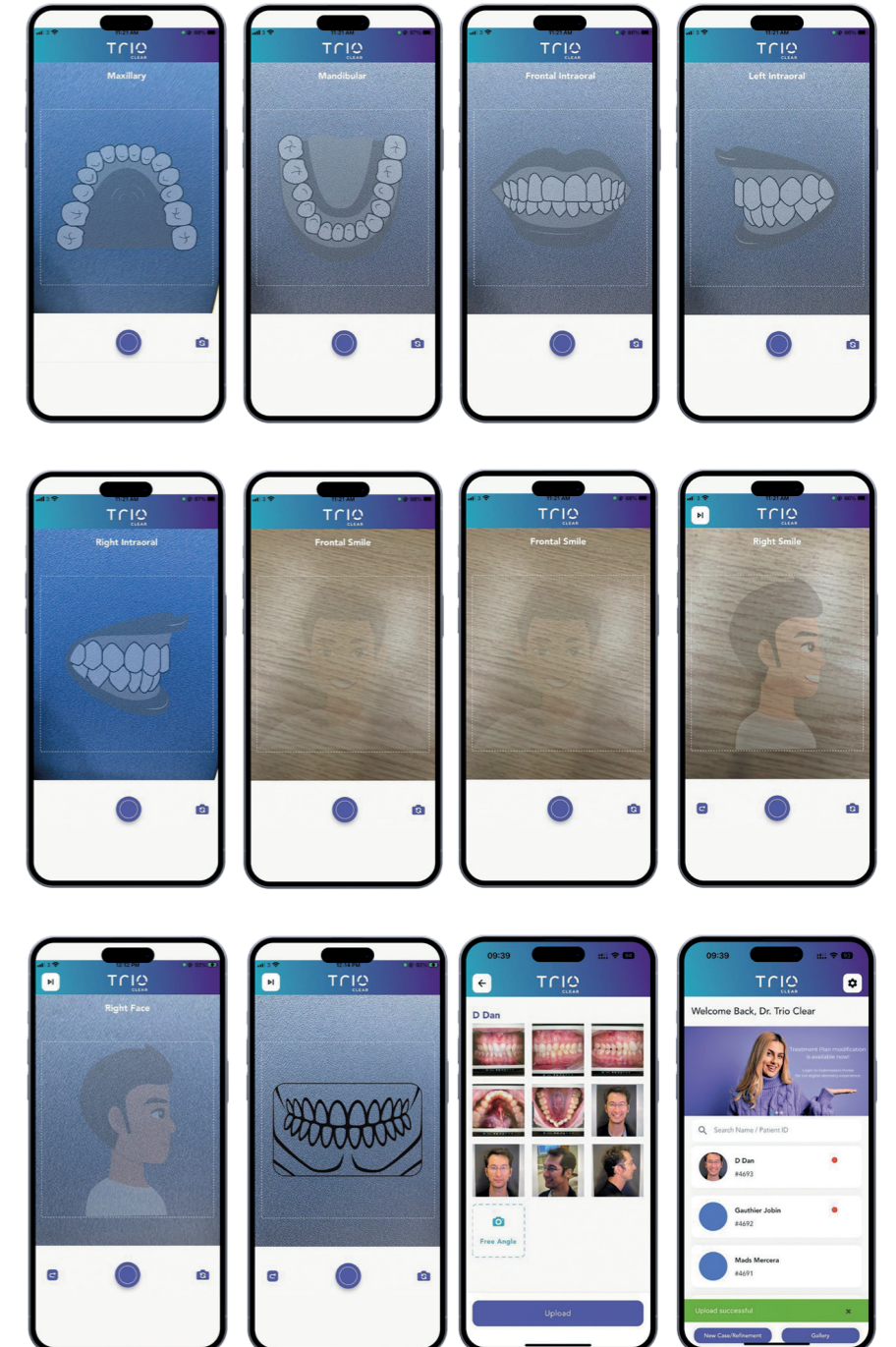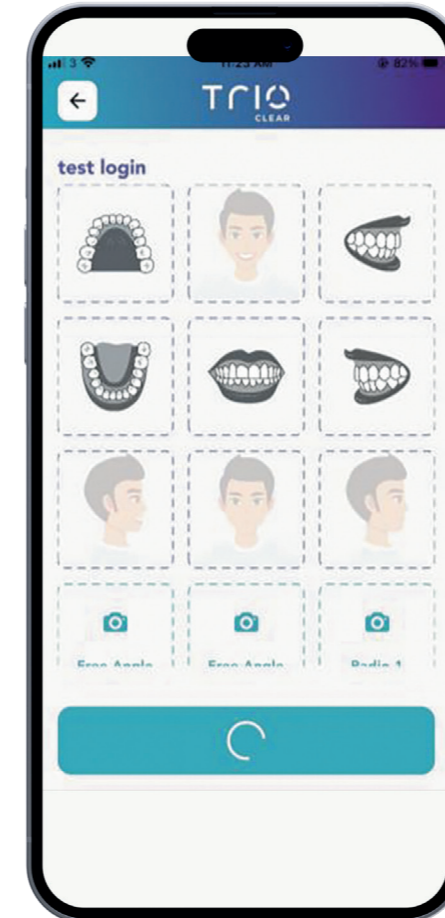

"Free angles" x 2 slots are added inside the flow for extra reference to doctors

"Radiographs" x 2 slots are added inside the flow can simply take photos and upload via TrioClear™ App

Final review


Upload successfully




NEW CASE REFINEMENT

Where can you find the photo folder?

SUBMISSION PORTAL CASE MANAGEMENT

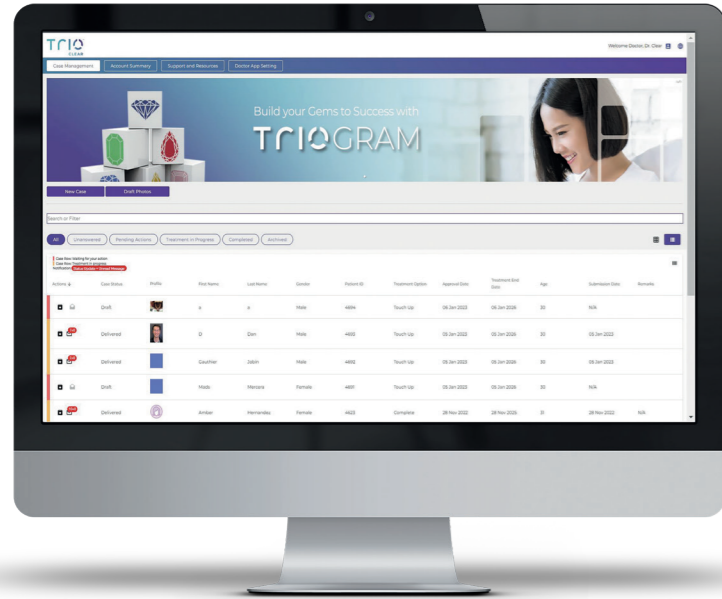


SUBMISSION PORTAL PHOTO FOLDER

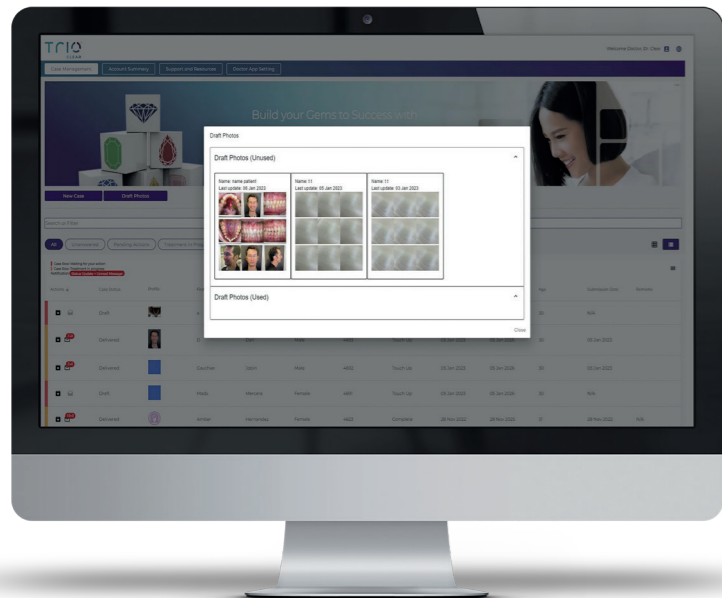

NEW CASE REFINEMENT


Where can you select the photos when creating case?

SUBMISSION PORTAL CREATE NEW CASE/ REFINEMENT

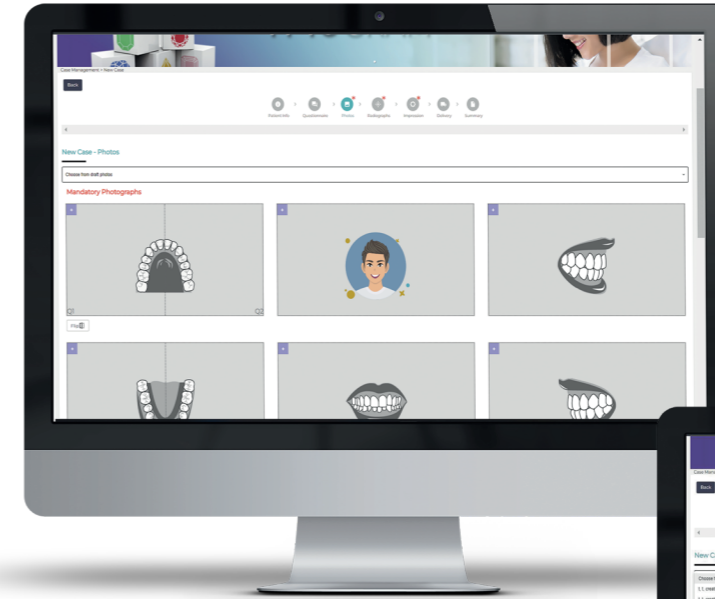


SUBMISSION PORTAL CHOOSE THE PATIENT NAME
INPUT FROM MOBILE APP

SUBMISSION PORTAL PHOTOS INSERT DIRECTLY

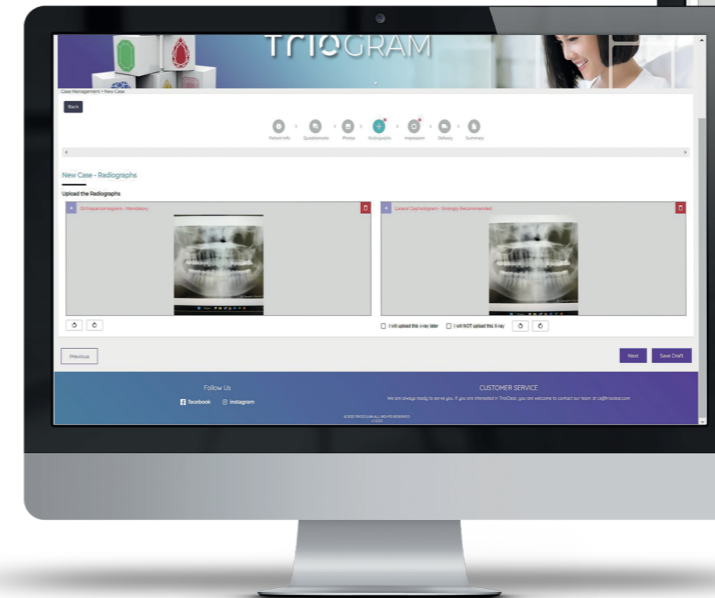

GALLERY


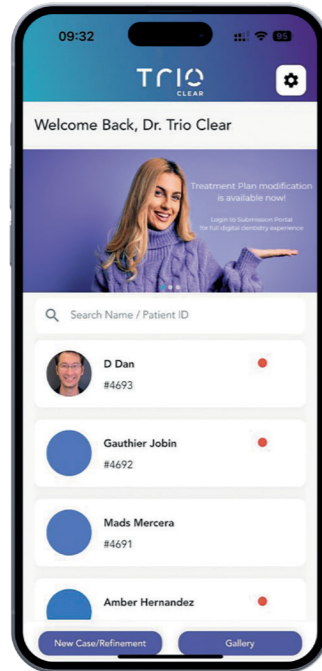


Choose "Gallery"




View the photo set taken by TrioClear™ App


You can still change the photo before using to submit cases/Refinement

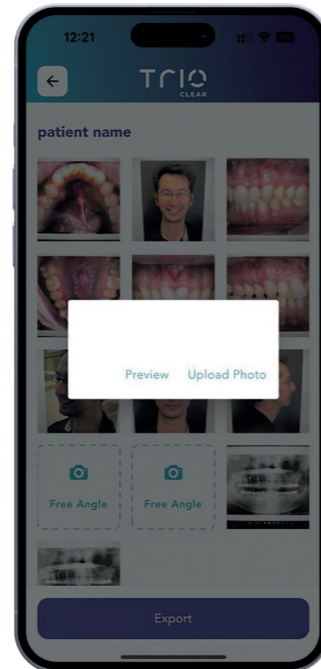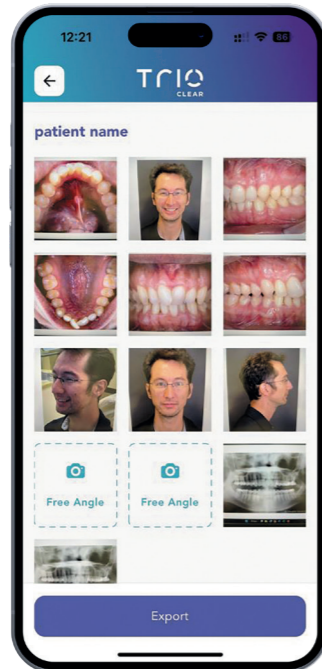


SETTINGS - MY APP ACCOUNT

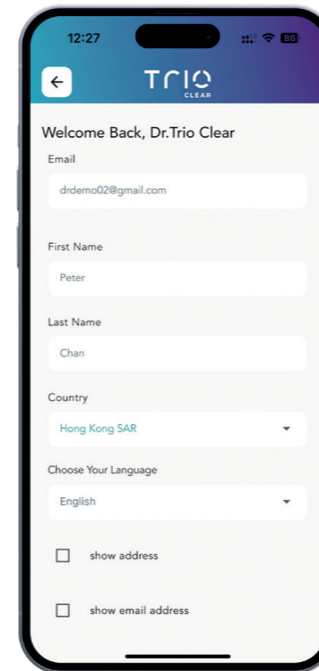

Change settings

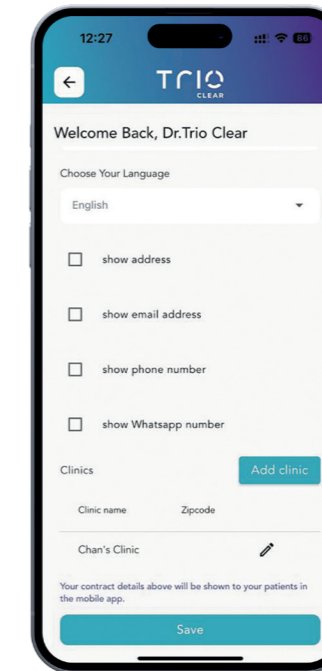

Options to display their contact method on patient interface

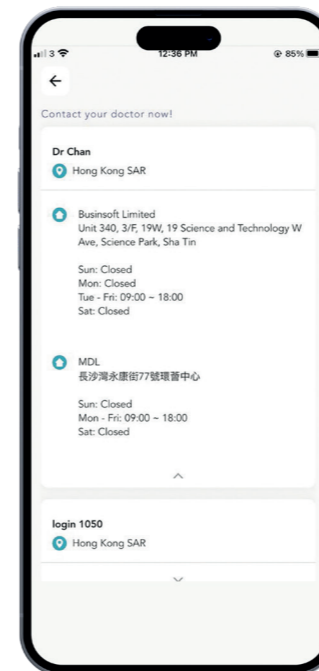

Need admin role to approve any adjustment on submission portal



MAKE YOUR PATIENTS **SMILE**

Follow us on **social media** TrioClear™ professionals 



Trees for All

