

INTRAORALE SCANBODIES

Cleaning instructions



Cleaning:

Automatic cleaning procedure based on Vario TD program:

- Cold water rinse 1 minute
- Cleaning at 55°C/131°F ($\pm 2^\circ\text{C}/35,6^\circ\text{F}$) for 5 minutes with cleaning agent neodisher® MediClean (0.5% v/v)
- Neutralization with cold water for 2 minutes
- Cold water rinse for 1 minute



Sterilization:

Steam sterilize the scanbodies using the cycle similar to instrumentation sterilization such as 4 minutes at 132°C/270°F.

** Alcohol and other ways of cleaning may have negative influence on the coating of the scanbody.*

