

# Medit Link account

Handleiding


# Account aanmaken

- 1 Ga naar [www.meditlink.com](http://www.meditlink.com) en klik op **“Don't have an account? Sign up”**.



The image shows the Medit Link login page. At the top, the text "Medit Link" is displayed in a large blue font. Below this, there are two input fields: "Email Address" and "Password". To the right of the password field is an eye icon for toggling visibility. Below the input fields, there is a checkbox labeled "Keep me logged in" and a link "Need help? -". A large blue button labeled "Log In" is centered below these elements. At the bottom of the page, there is a link "Don't have an account? Sign up" which is circled in red. At the very bottom, there is a language selection dropdown menu currently set to "ENGLISH(United States)".

- 2 Klik bij **“Clinic”** op **“Administrator”**.

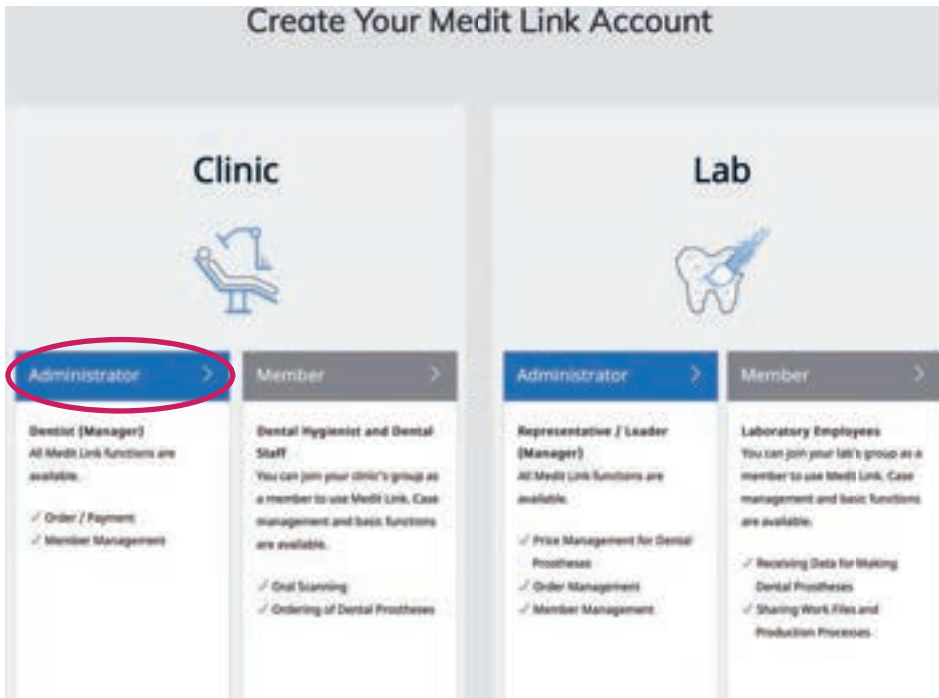

The image shows the "Create Your Medit Link Account" page. The page is divided into two main columns: "Clinic" and "Lab". Each column has a header with an icon (a dental chair for Clinic, a tooth for Lab) and a sub-header. Under each sub-header, there are two options: "Administrator" and "Member". The "Administrator" option under the "Clinic" column is circled in red. Below each option, there is a list of available functions. For the "Clinic Administrator" role, the functions listed are "Order / Payment" and "Member Management". For the "Clinic Member" role, the functions listed are "Oral Scanning" and "Ordering of Dental Prostheses". For the "Lab Administrator / Leader (Manager)" role, the functions listed are "Price Management for Dental Prostheses", "Order Management", and "Member Management". For the "Lab Member" role, the functions listed are "Receiving Data for Making Dental Prostheses" and "Sharing Work Files and Production Processes".

- 3 Vul alle velden in en klik op “Check Availability” en “Check” alvorens op “Register” te klikken.

The registration form contains the following fields and buttons:

- Clinic Name:** Tandartspraktijk X (with a checkmark icon) and a **Check Availability** button.
- Name:** John Doe
- Email Address:** info@tandartspraktijk.com (with a checkmark icon) and a **Check** button.
- Password:** [Redacted] and a **Check** button.
- Confirm Password:** [Redacted]
- Terms and Conditions:** Includes checkboxes for "Marketing and Promotion Agreement (Optional)", "Terms & Conditions (Mandatory)", and "Personal Information Collection Agreement (Mandatory)".
- Buttons:** "Back" and "Register" at the bottom.

Three callout boxes highlight the "Check Availability" button, the "Check" button, and the "Register" button. A separate callout box shows a confirmation message: "The email address is available." with a green checkmark and a "OK" button.

- 4 Controleer alle gegevens en klik op “OK”. Ga nu naar uw mailbox en zoek de e-mail van Medit Link. Controleer ook de spam folder indien de mail niet in Postvak In zit.

### Confirm Account Information


Clinic Name	Tandartspraktijk X
Name	John Doe
Email Address	info@tandartspraktijk.com

You will not be able to change your clinic name and email address once your registration is complete. Would you like to sign up with this information?


The screen displays an envelope icon and the following text:

- Please check your email to complete the registration.**
- Verification email has been sent to info@tandartspraktijk.com.
- You can complete the sign-up process once you check your email and click the verification link within 24 hours.
- Email Verification Time**  
Within 24 hours after signing up
- Your email address must be verified within 24 hours to protect your valuable personal information. If not, the information, such as email address and organization name you entered when signing up, will be initialized.
- At the bottom, there is a note: "If you entered the wrong email address, please proceed to sign-up again." and a "Sign up" button.

- 5 Open de e-mail en klik op **“Verify Email Address”**.

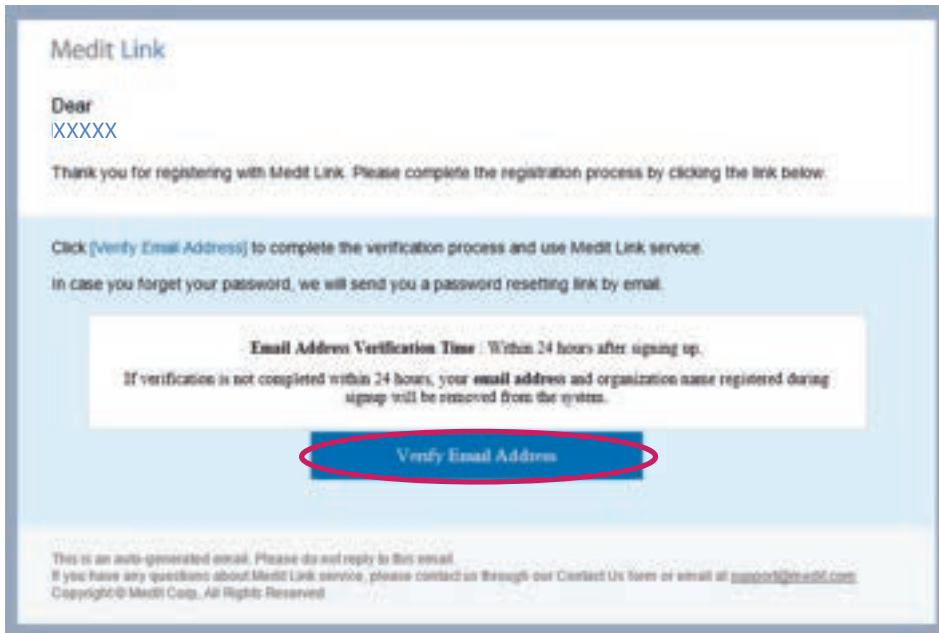


- 6 U werd nu doorverwezen via de webbrowser waar u op **“Log In Now”** klikt.

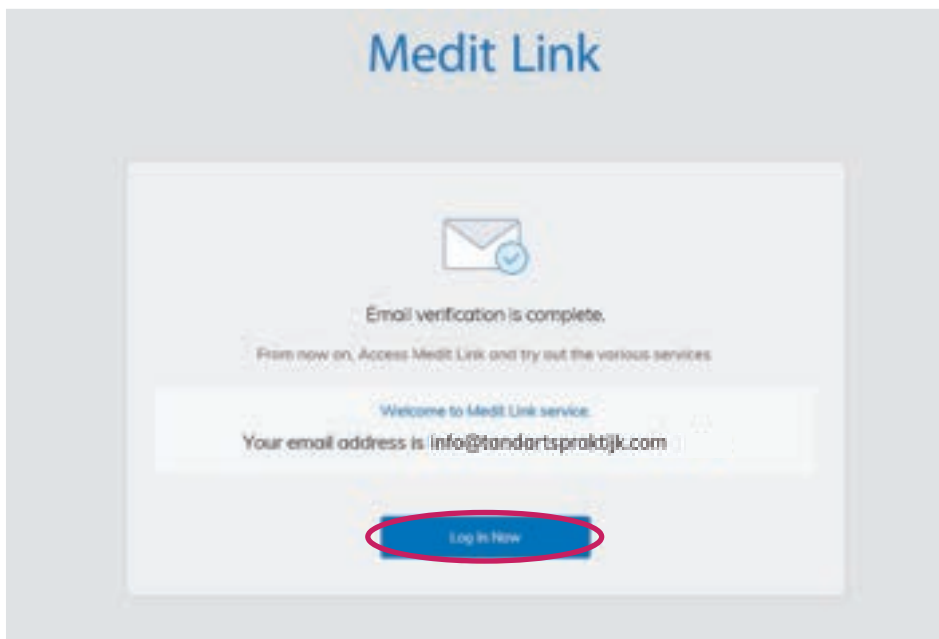

- 7 Vul uw e-mailadres en wachtwoord in en vink eventueel **“Keep Me Logged In”** aan, klik vervolgens op **“Log In”**.



8 Klik op "Let's Set Up Your Account".

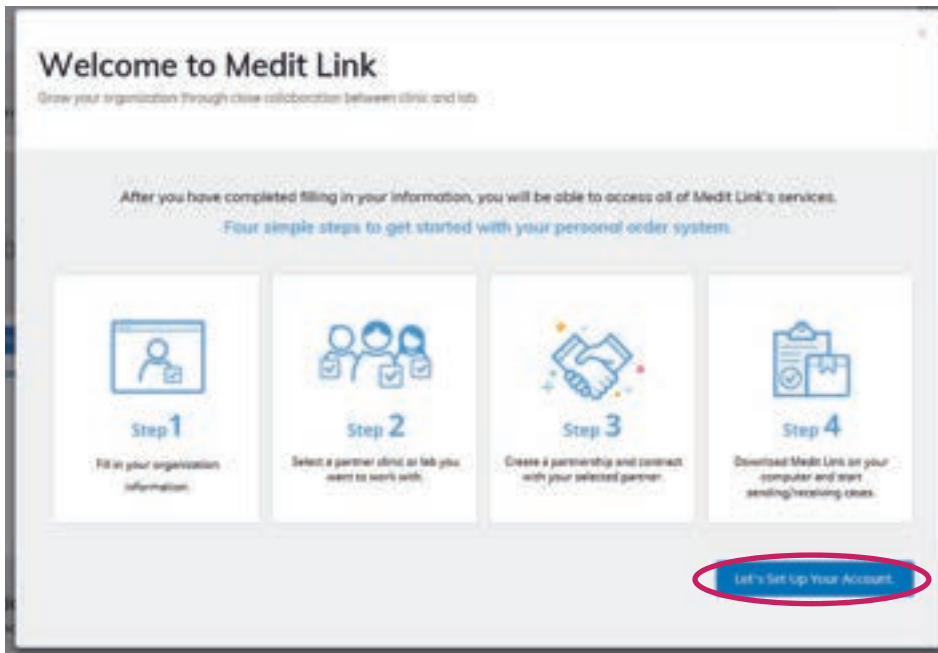

9 Vul uw adres gegevens in en klik op "Save".

Welcome to Medit Link  
Grow your organization through close collaboration between clinic and lab.

Address Information (Mandatory)  
Please enter your organization's address correctly. This information is shared with partners.

Address1 Flemingweg 14  
(Street Address, P.O. Box, Company Name, etc.)  
Please enter the correct address in order for labs to check your company's location on the map.

Address2 (Optional)  
(Apartment Suite, Unit, Building, Floor, etc.)

Zip/Postal Code 2408 City Alphen aan den Rijn


State/Province Vlaams-Brabant Country The Netherlands

Office Phone +31 172 653 300 Mobile Phone +31

This information is shared with the connected vendors for the purpose of product shipping.

Save Changes Save

10 Klik op "OK".

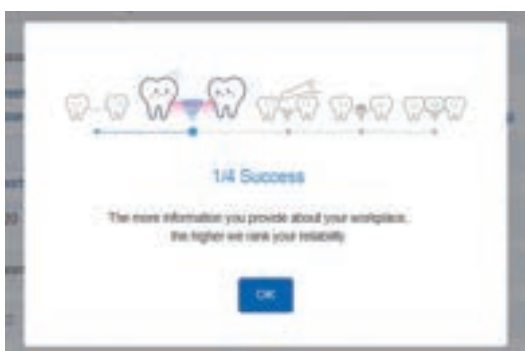



11

“Organization Information” is optioneel en kan u overslaan door op “Skip” te klikken. Indien u deze informatie wel wenst in te vullen kan u dit uiteraard doen en klikt u op “Save”. Deze info kan zichtbaar zijn voor de toekomstige Medit Partners.

The screenshot shows the 'Organization Information' step in the Medit Link setup process. At the top, there is a progress bar with four steps: 'Address Information' (checked), 'Organization Information' (checked), 'Scanner Information' (unchecked), and 'Additional Information' (unchecked). Below the progress bar, the title 'Organization Information (Optional)' is displayed, followed by the instruction 'Please provide information about your organization. This information is required for activating your organization in Medit Link.' The form contains several fields: 'Organization' (text input), 'Year Established' (dropdown menu), 'Business Hours' (text input with a time range selector), 'Business Registration' (text input), 'Business Registration' (checkbox with a note: 'Please enter your business number correctly. Make sure that the organization registered in Medit Link is identical to the Business Registration.'), 'Business Registration' (text input), and 'Website URL' (text input). At the bottom left, there is a 'Skip' button circled in red, and at the bottom right, there is a 'Save' button circled in blue.

12

“Scanner Information” kan u ook overslaan door op “Skip” te klikken. Uw toestel wordt geregistreerd bij de eerste ingebruikname.

The screenshot shows the 'Scanner Information' step in the Medit Link setup process. At the top, there is a progress bar with four steps: 'Address Information' (checked), 'Organization Information' (checked), 'Scanner Information' (checked), and 'Additional Information' (unchecked). Below the progress bar, the title 'Scanner Information' is displayed, followed by the instruction 'You can manage your equipment information in the area below.' The form contains a text input field for 'Model & Purchased Year' and three dropdown menus for 'Select Company', 'Select Model', and 'Select Year'. An 'Add Scanner' button is located at the bottom right of the form. At the bottom left, there is a 'Skip' button circled in red, and at the bottom right, there is a 'Save' button circled in blue.

- 13 Als laatste kan u optioneel extra informatie invullen.  
U kan ook niks invullen en meteen op "Save" klikken.


The screenshot shows the 'Additional Information' section of the Medit Link registration process. At the top, a progress bar indicates that 'Address Information', 'Organization Information', and 'Scanner Information' are completed, while 'Additional Information' is the current step. Below the header, a paragraph states: 'The information provided will be used to optimize Medit Link to meet your needs. The information will be secure and not shared with third parties.'

The form contains several input fields:

- Clinic Size (No. of Employees): Select
- No. of Dentists: Select
- No. of Dental Chairs in the Clinic: Select
- Type of Clinic: Select
- Portion of Work Done Using Digital Impressions: Slider (0% to 100%)
- Portion of Prosthodontics Ordered Overseas: Slider (0% to 100%)
- How many labs are you working with?: Select
- Average Delivery Time for Prosthodontics Ordered to External Lab: Digital (Select) Days, Analogue (Select) Days
- If you have an inhouse lab in the clinic, what CAD/CAM system do you use?: Select
- Do you have a CT machine in the clinic?: Select
- Which implant brand do you use?: Select
- Patient Management Software in Use: Text input field


A red circle highlights the 'Save' button at the bottom right of the form.

- 14 U heeft nu succesvol uw Medit Link account aangemaakt!

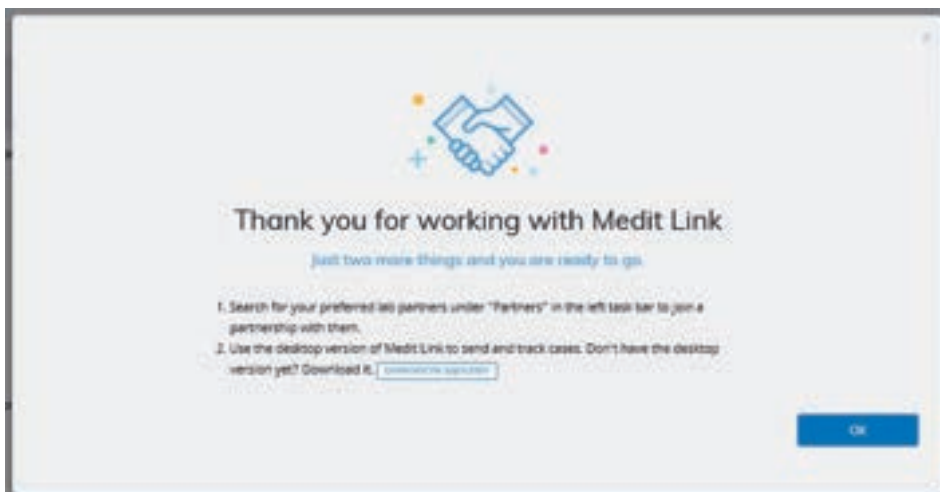


# Connectie met Elysee Dental



- 15 Log in met uw Medit Link account op [www.meditlink.com](http://www.meditlink.com) en klik op **“Partners”** en vervolgens op **“Search For Partners”**. Vul **“Elysee Dental NL”** in en druk op **Enter**.

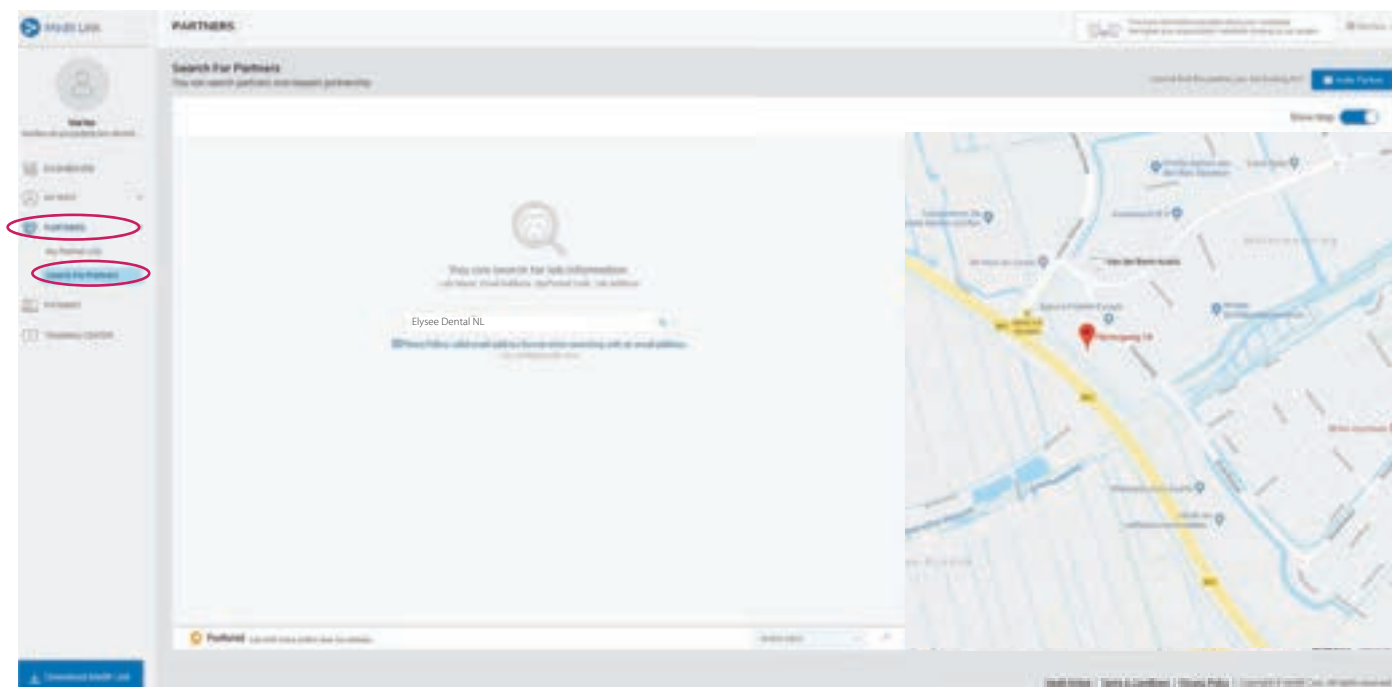

- 16 Klik op **“Request Partnership”** en vervolgens op **“Confirm”**.

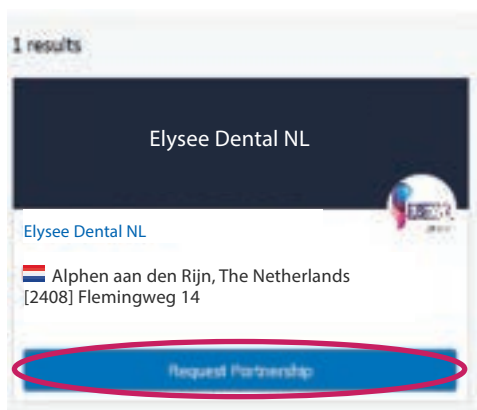


- 17 Klik tweemaal op **“OK”**.

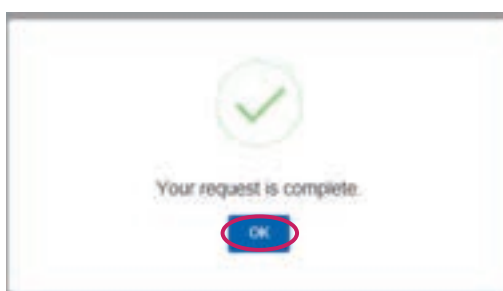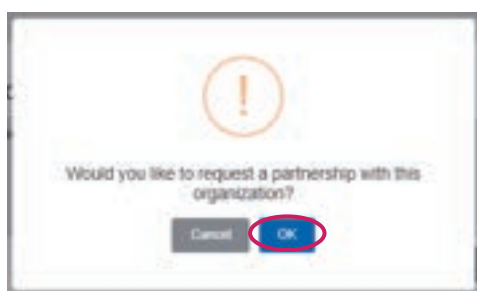




- 18 Wanneer wij uw aanvraag hebben goedgekeurd zal de status van **“Requesting”** naar **“Partnership: xxxx-xx-xx”** veranderen. Zodra dit gebeurd is zal Elysee Dental BE in de Medit Link software zichtbaar zijn in de lijst van beschikbare labo's.

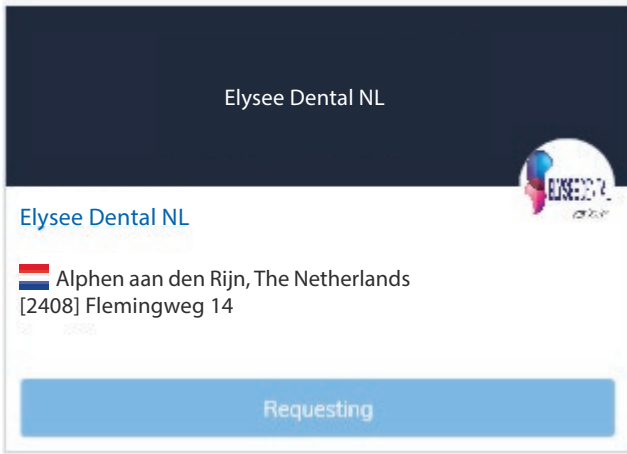


Elysee Dental NL

Elysee Dental NL

 Alphen aan den Rijn, The Netherlands  
[2408] Flemingweg 14

Requesting




Elysee Dental NL

Elysee Dental NL

 Alphen aan den Rijn, The Netherlands  
[2408] Flemingweg 14

Partnership : 2019-09-17

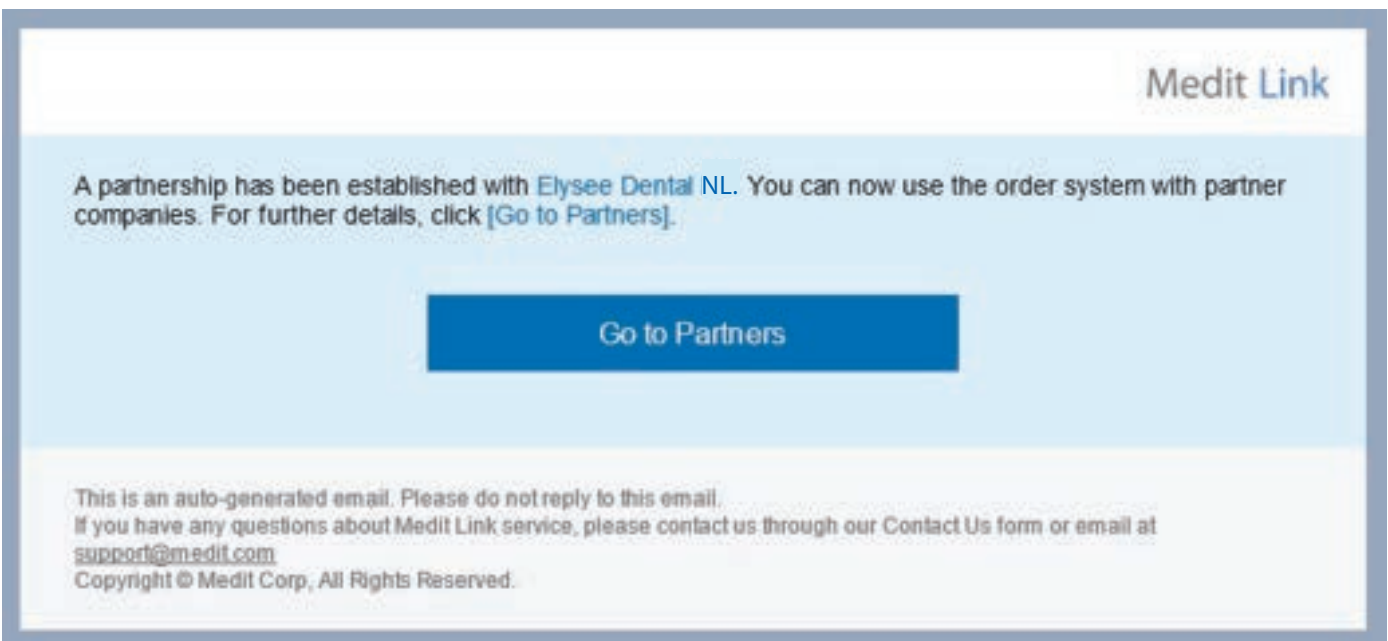

Medit Link

A partnership has been established with **Elysee Dental NL**. You can now use the order system with partner companies. For further details, click [\[Go to Partners\]](#).

[Go to Partners](#)

This is an auto-generated email. Please do not reply to this email.  
If you have any questions about Medit Link service, please contact us through our [Contact Us form](#) or email at [support@medit.com](mailto:support@medit.com)  
Copyright © Medit Corp, All Rights Reserved.

